

MODEL-med
international pty. ltd.

ABN 23 492 360 470

87 Argus St Cheltenham 3192
Melbourne Australia

Phone +61 (0)418 506 308 www.modelmed.com.au

MROP ADD-ON KIT
FOR SOPHIE'S MUM

Manual Removal of Placenta Kit Care and Handling

The Model-med MROP Kit is simple to use.

By simply placing the placenta inside the postpartum uterus and then placing both inside Sophie's Mum, with the support of the cradle, you can successfully simulate a Manual Removal of Placenta scenario




Lubricate the opening of the uterus with some water-based lubricant. Gently insert one hand inside the uterus, going past the cervix into the main body of the uterus.




Insert your other hand all the way into the uterus.




Stretch the uterus open.

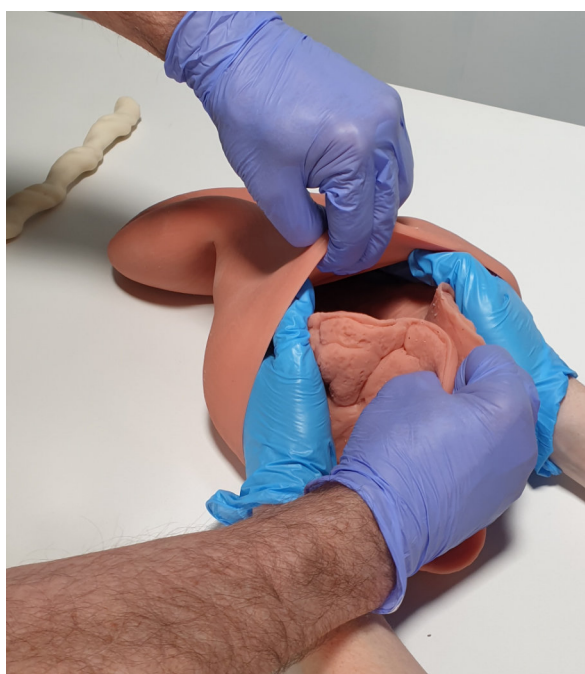

A colleague inserts the placenta into the lubricated uterus.

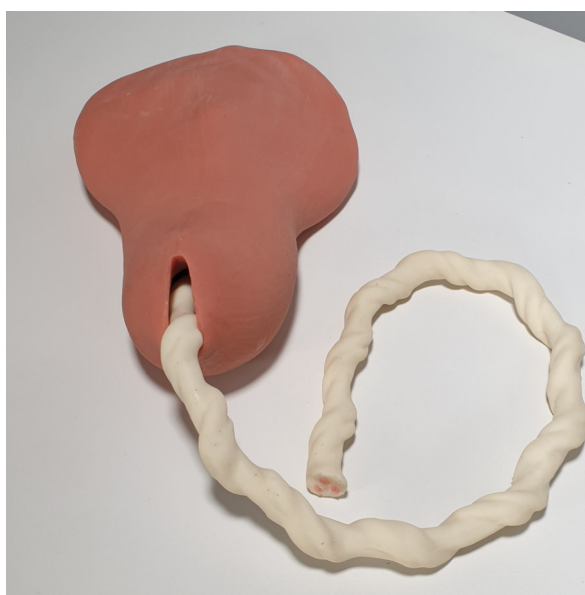


After withdrawing your hands you can now adjust the position of the placenta inside the uterus as desired.

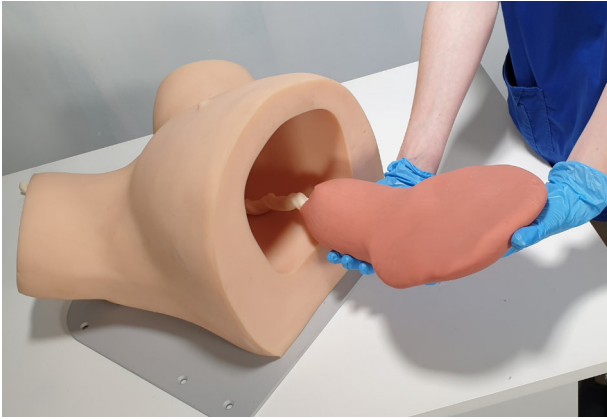


Place the uterus and placenta inside Sophie's Mum. Move the opening of the uterus to the opening of Sophie's Mum's vagina.



Both the outside of the uterus and the inside of Sophie's Mum should be dry and free from lubricant. That way the uterus will stay in place inside Sophie's Mum.

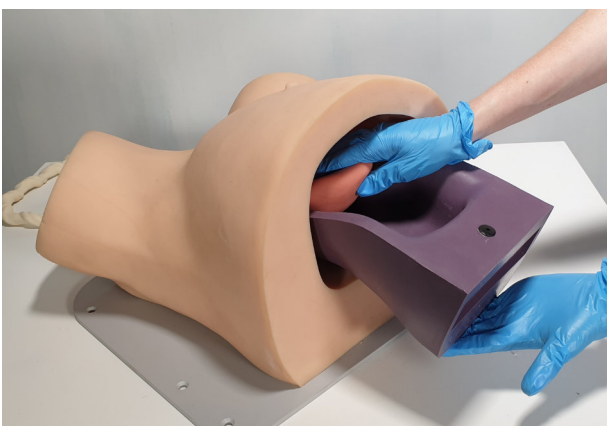

Insert the Cradle into the back of Sophie's Mum and underneath the uterus.

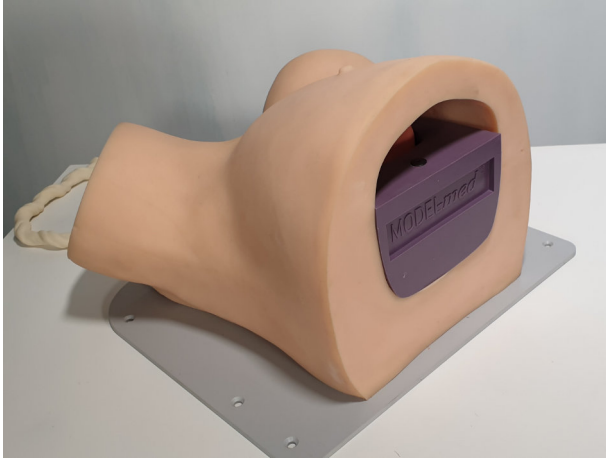

Adjust the uterus's position in the cradle until you are satisfied. The cradle will help lift and support the position of the uterus and help hold them in place during a simulation of MROP.



You are now ready to simulate a manual removal of the placenta. Remember to wear gloves and remove all jewellery and watches. Lubricate the hand and forearm well that you are going to insert into the vagina and uterus.

MODEL-med
international pty. ltd.

ABN 23 492 360 470

87 Argus St Cheltenham 3192
Melbourne Australia

Phone +61 (0)418 506 308 www.modelmed.com.au

Model-med International Pty Ltd Warranty

Our products are made from a soft, yet tough, material. They will last if they are treated like a person. That is, gently.

Model-med International Pty Ltd offers a one year warranty to the original purchaser from the date of purchase on our models when they are handled, operated and maintained in accordance with the printed instructions accompanying them. This Limited Warranty shall be limited to the repair or replacement of parts that prove defective under normal use and service and which on examination shall indicate, to Model-med's satisfaction, they are defective. If Model-med confirms the defect and approves the claim, Model-med will elect to repair or replace, or have parts repaired or replaced without charge by a nominated representative of Model-med 's choosing. We may ask for photographic evidence of the damage to the model. The Warranty does not cover any costs of shipping. Shipping costs will be paid by the Customer.

This Limited Warranty does not cover any failures or operating difficulties due to accident, abuse, misuse, alteration, misapplication, vandalism, improper maintenance, or failure to perform normal and routine maintenance.

We will repair a model if it fails within one year of purchase providing it has not been mistreated or abused. We define mistreated or abused in, but not limited to, the following ways:-

If the model is cut, punctured or sliced by any object, including but not limited to forceps, knives, or any other implements.

Damage caused by fingernails is not covered by this warranty.

If any lubricant other than a water-based lubricant is used on the model. Only recognised water-based surgical lubricants should be used.

If any chemicals other than those described in the care and handling documents are used on the skin, or if the chemicals described in the care and handling documents are used in quantities greater than those recommended.

Hot objects should not be placed on or around the model. We define hot as being above 50 degrees celcius.

If you believe you have a claim under warranty you must first contact your local distributor for instructions on how to proceed with the claim.

Model-med International Pty Ltd shall not be liable under this or any implied warranty for incidental or consequential damages.